

## CM 23714 – BACTEROIDES SELECTIVE SUPPLEMENT

### INTENDED USE

For selective isolation of Bacteriodes species.

### COMPOSITION

Ingredients	Concentration
Gentamicin	50mg

(Per vial sufficient for 500 ml medium)

### Applied in Medium




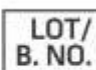








BACTEROIDESBILE ESCULIN AGAR BASE (BBE AGAR BASE)

### INSTRUCTION FOR USE

Rehydrate the contents of one vial aseptically with 5.0 ml sterile distilled water. Mix well and aseptically add to 500 ml sterile molten, cooled (45-50°C) BACTEROIDES BILE ESCULIN AGAR BASE (BBE AGAR BASE). Mix well and pour in sterile petri plates.

### STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative <small>MedWor GmbH Sachsenring 15 40102 Düsseldorf, Germany</small>	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

\*For LabUse Only

