

CM 23730 – CAMPYLOBACTER SUPPLEMENT V (BFCSA)

INTENDED USE

For selective isolation of Campylobacter species.

COMPOSITION

Ingredients	Concentration
Cefoperazone	16mg

(Per vial sufficient for 500 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of one vial aseptically with 2 ml sterile distilled water. Mix well and aseptically add the rehydrated contents to 500 ml sterile, molten, cooled (45-50°C) BLOOD FREE CAMPYLOBACTER SELECTIVITY AGAR BASE (ISO 10272-1&2:2017). Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Seckestraße 11C 40113 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only