

CM 23740 – CEFOTAXIME SUPPLEMENT

INTENDED USE

For selective isolation of Methicillin Resistant *Staphylococcus aureus* from clinical specimens.

COMPOSITION

Ingredients	Concentration
Cefotaxime	3mg

(Per vial sufficient for 500ml medium)

Applied in Medium

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial with 5 ml of sterile distilled water and aseptically add to 500 ml of sterile molten - CHROMOGENIC MeReSA AGAR BASE / MeReSA AGAR BASE. This supplement can either be used individually or in combination with MeReSa Selective Supplement for more selectivity. Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MediStar GmbH Buckhorn 10, 48147 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only