

## CM 23749 – CHROMOGENIC CANDIDA SELECTIVE SUPPLEMENT

### INTENDED USE

For isolation and identification of Candida species from mixed cultures.

### COMPOSITION

| Ingredients | Concentration |
|-------------|---------------|
| Gentamicin  | 50mg          |

(Per vial sufficient for 500 ml medium)

### INSTRUCTION FOR USE

Rehydrate the content of 1 vial with 5 ml of sterile distilled water. Mix well and aseptically add to 500 ml of sterile, molten, cooled (45-50°C) CHROMOGENIC CANDIDA DIFFERENTIAL AGAR MODIFIED. Mix well and pour into sterile petri plates.

### STORAGE

Vial should be stored in sealed container at 2-8°C.

|                                                                                                                                                |                                                                                                                                                                                                                     |                                                                                                                         |                                                                                                                                     |                                                                                                                                   |                                                                                                              |
|------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------|
| <br><b>GMP</b><br>Good Manufacturing<br>Practices Certified | <br><b>IVD</b><br>For In Vitro Diagnostic Use                                                                                    | <br><b>QTY.</b><br>Quantity          | <br><b>LOT/<br/>B. NO.</b><br>Lot / Batch Number | <br><b>REF</b><br>Catalogue Number           | <br><b>Manufacturer</b> |
| <br><b>Temperature Unit</b>                                 | <br><b>EC REP</b><br>Authorized Representative<br><small>MedWet GmbH<br/>Seidenstrasse 15,<br/>40142 Düsseldorf, Germany</small> | <br><b>CE</b><br>European Conformity | <br><b>QR<br/>Code</b>                           | <br><b>Consults Instructions<br/>for Use</b> | <br><b>Best Before</b>  |

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

\*For Lab Use Only