

CM 23750 – CHROMOGENIC COLISTIN RESISTANT SELECTIVE SUPPLEMENT

INTENDED USE

For isolation & differentiation of colistin-resistant pathogens.

COMPOSITION

Ingredients	Concentration
Colistin sulphate	8.0mg

(Per vial sufficient for 1000ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 10 ml sterile distilled water. Mix gently to dissolve the contents completely. Aseptically add to 1000 ml of sterile, molten, cooled (45-50°C) CHROMOGENIC COLISTIN RESISTANT AGAR BASE. Mix well and pour into sterile Petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Seckestraße 11C 48153 Münster, Germany	 European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only