

CM 23756 – CHROMOGENIC MeReSa SELECTIVE SUPPLEMENT

INTENDED USE

For the selective isolation of Methicillin Resistant Staphylococcus aureus from clinical specimens.

COMPOSITION

Ingredients	Concentration
Methicillin	2mg

(Per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of one vial with 5 ml of sterile distilled water and aseptically add to 500 ml of sterile, molten, cooled (45-50°C) CHROMOGENIC MeReSa AGAR BASE. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Seestraße 115 40113 Düsseldorf, Germany	 CE European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only