

CM 23766 – DERMATO SUPPLEMENT

INTENDED USE

For the determination of tryptophan-deaminase activity.

COMPOSITION

Ingredients	Concentration
Cycloheximide	250mg
Chlortetracycline	50mg
Gentamicin	50mg

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial with 5 ml of 50% v/v acetone. Mix well and aseptically add it to 500 ml of sterile, molten, cooled (45-50°C) D.T.M. AGAR BASE. Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative MedWet GmbH Buckhorn 15, 40142 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.