

CM 23768 – DMACA REAGENT (10 ml/vl)

INTENDED USE

Enumeration and differentiation of enteric pathogens in urinary tract infections.

COMPOSITION

Ingredients	Concentration
p-Dimethylaminocinnamaldehyde	1.000gm
Hydrochloric acid (concentrated)	1.000ml
Distilled water	99.000ml

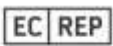
(per vialsufficientfor500mlmedium)

INSTRUCTION FOR USE

With a platinum wire loop, plastic loop or glass rod spread suspected colony from CHROMOGENIC UTI AGAR, MODIFIED plate on filter paper dipped in DMACA Reagent. Observe for appearance of bluish-purple colour within 10-30 seconds.

STORAGE

Vialsshouldbe stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Mallat GSKH Bachman 10, 40142 Munich, Germany	 CE European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For LabUse Only