

CM 23782 – G. VAGINALIS SELECTIVE SUPPLEMENT

INTENDED USE

For selective isolation of Gardnerella vaginalis.

COMPOSITION

Ingredients	Concentration
Gentamicin	2mg
Nalidixic acid	15mg
Amphotericin B	1mg

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml of sterile distilled water. Mix well and aseptically add it to 500 ml of sterile, molten, cooled (45-50°C) Columbia Blood Agar Base / COLUMBIA BLOOD AGAR BASE (VEG.) / COLUMBIA BLOOD AGAR BASE W/ 1% AGAR along with 5% v/v sterile defibrinated sheep blood. Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Mallin GmbH Buckhorn 10, 40142 Essen, Germany	 CE European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only

