

## CM 23784 – GBS SUPPLEMENT

### INTENDED USE

For selective isolation and presumptive identification of group B Streptococci.

### COMPOSITION

| Ingredients   | Concentration |
|---------------|---------------|
| Metronidazole | 5mg           |
| Gentamicin    | 1mg           |

(per vial sufficient for 500 ml medium)

### INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml of sterile distilled water. Mix well and add it to 475 ml sterile, molten, cooled (45-50°C) GBS MEDIUM BASE along with 25 ml inactivated Horse serum Mix well.

### STORAGE

Vials should be stored in sealed container at 2-8°C.

|                                                                                                                                         |                                                                                                                                                                                           |                                                                                                            |                                                                                                                             |                                                                                                                           |                                                                                                       |
|-----------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------|
| <br>GMP<br>Good Manufacturing<br>Practices Certified | <br>IVD<br>For In Vitro Diagnostic Use                                                                 | <br>QTY.<br>Quantity    | <br>LOT/<br>B. NO.<br>Lot / Batch Number | <br>REF<br>Catalogue Number          | <br>Manufacturer |
| <br>Temperature Unit                                 | <br>EC REP<br>Authorized Representative<br>Mallin+Gottlieb<br>BioScience 10,<br>40143 Krefeld, Germany | <br>European Conformity | <br>QR<br>Code                           | <br>Consults Instructions<br>for Use | <br>Best Before  |

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

\*For Lab Use Only