

CM 23793 – HAEMOPHILUS GROWTH SUPPLEMENT

INTENDED USE

For cultivation of Haemophilus influenzae.

COMPOSITION

Ingredients	Concentration
NAD	7.500mg
Haematin	7.500mg

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
(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of one vial aseptically with 5.0 ml sterile distilled water. Mix well and aseptically add to 500 ml sterile, molten, cooled (45-50°C) HAEMOPHILUS TEST AGAR BASE. Mix well and aseptically pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative Mallatier GmbH Buckhorn 10, 40143 Essen, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only