

CM 23800 – KLEBSIELLA SELECTIVE SUPPLEMENT

INTENDED USE

For selective isolation and easy detection of Klebsiella species from water and other sources.

COMPOSITION

Ingredients	Concentration
Carbenicillin	25mg

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml of sterile distilled water. Mix well and aseptically add it to 500 ml of sterile, molten, cooled (45-50°C) MacCONKEY AGAR, MODIFIED / M-KLEB AGAR BASE. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Südstraße 115 48143 Münster, Germany	 European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only