

## CM 23824 – LINCO T SUPPLEMENT (LINCOMYCIN-COLISTIN-AMPHOTERICIN-B TRIMETHOPRIM)

### INTENDED USE

For selective isolation and cultivation of Gonococci.

### COMPOSITION

| Ingredients          | Concentration |
|----------------------|---------------|
| Trimethoprim lactate | 3.250 mg      |
| Colistin             | 3.000 mg      |
| Lincomycin           | 0.500mg       |
| Amphotericin B       | 0.500mg       |

(Per vial sufficient for 500 ml medium)

### INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 10 ml of sterile distilled water. Mix gently, avoiding frothing to dissolve and aseptically add it to 240 ml of sterile, molten, cooled (45-50°C), GC Agar Base (TM 116) / GC HiVeg™ Agar Base (TMV 116) along with 250 ml of sterile 2% w/v Haemoglobin solution. Also add sterile rehydrated contents of 1 vial of Yeast Autolysate Supplement (TS 023). Mix gently and pour into sterile petri plates

### STORAGE

Vial should be stored in sealed container at 2-8°C.

|                                                                                                                                         |                                                                                                                                                                                                         |                                                                                                            |                                                                                                                             |                                                                                                                           |                                                                                                       |
|-----------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------|
| <br>GMP<br>Good Manufacturing<br>Practices Certified | <br>IVD<br>For In Vitro Diagnostic Use                                                                               | <br>QTY.<br>Quantity    | <br>LOT/<br>B. NO.<br>Lot / Batch Number | <br>REF<br>Catalogue Number          | <br>Manufacturer |
| <br>Temperature Unit                                 | <br>EC REP<br>Authorized Representative<br><small>MedNet GmbH<br/>Barkstrasse 10,<br/>48163 Münster, Germany</small> | <br>European Conformity | <br>QR<br>Code                           | <br>Consults Instructions<br>for Use | <br>Best Before  |

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

\*For Lab Use Only

