

CM 23838 – MEROPENEM SUPPLEMENT

INTENDED USE

For enrichment of Enterococci.

COMPOSITION

Ingredients	Concentration
Meropenem	1mg

(Per vial sufficient for 500 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial with 5 ml of sterile distilled water and aseptically add to 1000 ml of sterile molten, cooled (45-50°C) VRE AGAR BASE (VANCOMYCIN RESISTANT ENTEROCOCCI AGAR) along with 2 vials of Vancomycin Supplement or aseptically add 2 vials to 1000 ml of VRE BROTH BASE (VANCOMYCIN RESISTANT ENTEROCOCCI BROTH). Mix well and dispense as desired.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MediStar GmbH Buckhorn 10, 48147 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only