

CM 23847 – MUG SUPPLEMENT (50 MG PER VIAL)

INTENDED USE

For measuring β -glucuronidase activity, for rapid and sensitive identification of Escherichia coli.

COMPOSITION

Ingredients	Concentration
4-Methylumbelliferyl b-D-Glucuronide(MUG)	50mg

(Per vial sufficient for 500/1000 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of one vial aseptically with 5 ml methanol. One vial is sufficient for 500 ml agar media or 1000 ml of broth media. Add the contents of 1 vial to 1000 ml of sterile, molten, cooled (45-50°C) M-FC BASAL MEDIUM. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Seestraße 11 48153 Münster, Germany	 CE European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only