

CM 23849 – MYCOPLASMA ENRICHMENT SUPPLEMENT

INTENDED USE

For isolation of Mycoplasma.

COMPOSITION

Ingredients	Concentration
Horse serum	20ml
Yeast extract (25%w/v solution)	10ml
Thallium acetate	25mg
Penicillin G	20000IU

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
(Per vial sufficient for 1000ml medium)

INSTRUCTION FOR USE

Aseptically add the contents of 1 vial to 70 ml of sterile, molten, cooled (45-50°C) MYCOPLASMA AGAR BASE (PPLO AGAR BASE) / MYCOPLASMA BROTH BASE W/O CV (PPLO BROTH BASE W/O CV) / MYCOPLASMA BROTH BASE W/ CV (PPLO BROTH BASE W/ CV). For MYCOPLASMA BROTH BASE W/ CV M268, add 0.285 ml of 1% Potassium Tellurite along with Mycoplasma Enrichment Supplement. Mix well and dispense as desired.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Mallier GmbH Buckhorn 10 40143 Aachen, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only

