

CM 23850 – MYCOPLASMA SELECTIVE SUPPLEMENT

INTENDED USE

Supplement for the isolation of Mycoplasma.

COMPOSITION

Ingredients	Concentration
Dextrose	1.80grams
Yeast extract	0.36 grams
Thallium acetate	0.036grams
Penicillin G	3,60,000 units

(Per vial sufficient for 1000ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of one vial with 5 ml sterile distilled water and aseptically add to 1000 ml of TRYPTOSE SERUM AGAR BASE / TRYPTOSE SERUM BROTH BASE (MODIFIED NEWINGS TRYPTOSE BROTH BASE) along with 130ml of pig serum, and inactivate at 56°C for not more than 30 minutes.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative MedWet GmbH Buckenhof 15 48143 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only