

CM 23852 – NALIDIXIC SELECTIVE SUPPLEMENT

INTENDED USE

For selective isolation of *Pseudomonas aeruginosa* from clinical specimens.

COMPOSITION

Ingredients	Concentration
Nalidixic acid	15mg

(per vial sufficient for 1000 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 5 ml of sterile distilled water. Mix well and aseptically add to 1000 ml of sterile, molten, cooled (45-50°C) CETRIMIDE AGAR BASE (W 1.3% AGAR) / CETRIMIDE AGAR BASE / CETRIMIDE AGAR BASE (VEG.). Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Mueller GmbH Buckhorn 10, 48161 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only