

CM 23855 – NEOMYCIN SUPPLEMENT

INTENDED USE

For selective isolation of Streptococcus species.

COMPOSITION

Ingredients	Concentration
Neomycin sulphate	30mg

(per vial sufficient for 940 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 10 ml of sterile distilled water. Mix gently to dissolve. Aseptically add the contents to 940 ml of sterile, molten, cooled (45-50°C) ANAEROBIC BLOOD AGAR BASE / PAGANO LEVIN BASE along with 50 ml of sterile defibrinated sheep blood. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedNet GmbH Buckhorn 10 40113 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only