

CM 23856 – NON SPORE ANAEROBIC SUPPLEMENT

INTENDED USE

For selective isolation or cultivation of non-sporing anaerobic bacteria.

COMPOSITION

Ingredients	Concentration
Sodium pyruvate	500mg
Menadione	0.250mg
Ferric pyrophosphate, soluble	2.500mg
Nalidixic acid	5mg

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 10 ml sterile distilled water. Mix well and aseptically add to sterile, molten, cooled (45-50°C) 490 ml of WILKINS CHALGREN ANAEROBIC AGAR BASE / WILKINS CHALGREN ANAEROBIC BROTH BASE, enriched with 5% v/v sterile defibrinated blood. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative MedWet GmbH Buckhorn 15, 40142 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only