

CM 23880 – PYR REAGENT (10 ml/vl)

INTENDED USE

For detection of pyrrolidonyl arylamidase enzyme activity of β -hemolytic Streptococci.

COMPOSITION

Ingredients	Concentration
N,N -Dimethylaminocinnamaldehyde	1.0gm
Hydrochloric acid (concentrated)	1.0ml
Distilled water	99.0ml

INSTRUCTION FOR USE

Inoculate test culture on PYR Agar and incubate at 35-37°C for 18-24 hours.

Add 1 drop of PYR Reagent directly to suspected colony.

Observe for colour change after 2 minutes.

Formation of red colour indicates PYR positive organism.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative MedWet GmbH Seckestraße 10, 48143 Münster, Germany	 European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only