

CM 23889 – SELECTIVE SUPPLEMENT FOR MRSA

INTENDED USE

Recommended for the selective detection of methicillin-resistant *Staphylococcus aureus* from clinical specimens.

COMPOSITION

Ingredients	Concentration
Cefoxitin	10.0mg
Colistin	10.0mg
Amphotericin B	10.0mg

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
(per vial sufficient for 1000 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 1 ml of 0.2 N Sodium hydroxide, further add 9 ml of sterile distilled water. Mix well and aseptically add to 1000 ml of sterile, molten, cooled (45-50°C) SELECTIVE BROTH FOR MRSA. Mix well and pour into sterile Petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative Mallatier GmbH Buckhorn 16 40162 Düsseldorf, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only