

CM 23892 – SODIUM BICARBONATE SOLUTION (20ml/vl)

INTENDED USE

For selective isolation of Enterococcus species.

COMPOSITION

Ingredients	Concentration
Sodium bicarbonate (10%w / v)	20ml

(per vial sufficient for 980ml medium)

INSTRUCTION FOR USE

Warm up the refrigerated solution to room temperature. Add the contents of 1 vial aseptically with 980 ml sterile, molten, cooled (45-50°C) GTC Agar Base M922 along with 2 vials of GTC Supplement. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Seckestraße 11C 48153 Münster, Germany	 European Conformity	 QR Code	 Consults instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only