

CM 23893 – STAPH STREPTO SUPPLEMENT

INTENDED USE

For selective isolation of Staphylococci and Streptococci.

COMPOSITION

Ingredients	Concentration
Nalidixic acid	1mg
Colistin sulphate	1mg

(per vial sufficient for 500 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml of sterile distilled water. Mix well and aseptically add it to 500 ml of sterile, molten, cooled (45-50°C) COLUMBIA BLOOD AGAR BASE / COLUMBIA BLOOD AGAR BASE (VEG.) along with 25 ml of sterile defibrinated horse blood, COLUMBIA BLOOD AGAR BASE W/ 1% AGAR. Mix gently and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWor GmbH Buckhorn 16, 40162 Düsseldorf, Germany	 CE European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only