

CM 23899 – TDA REAGENT (10 ml/vl)

INTENDED USE

For determination of tryptophan-deaminase activity.

COMPOSITION

Ingredients	Concentration
Ferric chloride	10.000gm
Distilled water	100.000ml

INSTRUCTION FOR USE

Add 1- 2 drops of TDA reagent directly on the suspected colony from CHROMOGENIC UTI AGAR, MODIFIED. Observe for the appearance of dark brown colouration around the colony within 10 – 30 seconds to confirm a positive reaction.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MediStar GmbH Buckhorn 16, 48143 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For LabUse Only