

CM 23905 – TRICHOMONAS SELECTIVE SUPPLEMENT I

INTENDED USE

For selective isolation of Trichoderma species from clinical specimens.

COMPOSITION

Ingredients	Concentration
Streptomycin	500mg
Penicillin G	125000Unit

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
(per vial sufficient for 500 ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 5 ml of sterile distilled water. Mix well and avoid frothing of solution. Aseptically add the contents to 475 ml of sterile, molten, cooled (45-50°C) KUPFERBERG TRICHOMONAS BROTH BASE (TRICHOMONAS BROTH BASE, KUPFERBERG) along with 25 ml sterile beef or human serum. Mix well and dispense as desired.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MediStar GmbH Buckhorn 16, 48161 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only