

CM 23906 – TRICHOMONAS SELECTIVE SUPPLEMENT II

INTENDED USE

For selective isolation of Trichoderma species.

COMPOSITION

Ingredients	Concentration
Streptomycin	250mg
Penicillin G	500000Unit

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 5 ml of sterile distilled water. Mix well, avoiding frothing of the solution. Aseptically add the contents of 1 vial to 460 ml of sterile, molten, cooled (45-50°C) TRICHOMONAS AGAR BASE along with 40 ml of inactivated Horse Serum. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Buckhorn 16, 48163 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only