

CM 23910 – UREA SOLUTION 5% (5 ml/vl)

INTENDED USE

For selective cultivation of Mycoplasma hominis and Ureaplasma urealyticum.

COMPOSITION

Ingredients	Concentration
Urea	0.250g
Distilled water	5ml

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
(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Warm up the refrigerated Urea solution to room temperature and aseptically add 1 vial to 425 ml of sterile, molten, cooled (45-50°C) UROGENITAL MYCOPLASMA MEDIUM BASE along with 1 vial of Vitamin amino Growth Supplement and 1 vial of Mycoplasma Urogenital Selective Supplement and 50 ml of Horse Serum. Mix well and dispense as desired.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MedWet GmbH Buckhorn 16, 48163 Münster, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only