

CM 23911 – V.C.N. SUPPLEMENT

INTENDED USE

For selective isolation of *Neisseria gonorrhoeae* and *Neisseria meningitidis*.

COMPOSITION

Ingredients	Concentration
Vancomycin	1.500mg
Colistin methane sulphonate	3.750mg
Nystatin	6250Units

INSTRUCTION FOR USE (per vial sufficient for 500ml medium)

Rehydrate the contents of 1 vial aseptically with 2 ml sterile distilled water. Mix well and aseptically add it to THAYER MARTIN MEDIUM BASE for 440 ml of medium aseptically add 50ml sterile lysed blood and one vial of V.C.N. Supplement along with one vial of Vitamino Growth Supplement, 2% Haemoglobin Solution (250ml) can be used instead of sterile lysed blood in 250ml of medium.

In GC AGAR BASE / GC AGAR BASE (VEG.) for 250 ml of can be used instead of sterile lysed blood in 250 ml of 2% Haemoglobin Solution and GC Supplement w/Antibiotics, one vial of Vitamino Growth Supplement w/

Antibiotics for additional selectivity. If desired V.C.N. Supplement can be used along with GC Supplement w/ Antibiotics

for additional selectivity.

In TRANSGROW MEDIUM BASE for 440 ml of medium aseptically add 50 ml of sterile 2% Haemoglobin Solution and one vial of V.C.N. Supplement along with one vial of Vitamino Growth Supplement.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 Good Manufacturing Practices Certified	 For In Vitro Diagnostic Use	 Quantity	 Lot / Batch Number	 Catalogue Number	 Manufacturer
 Temperature Unit	 Authorized Representative	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only

