

CM 23912 – V.C.N.T. SUPPLEMENT

INTENDED USE

For selective isolation of *Neisseria gonorrhoeae* and *Neisseria meningitidis*.

COMPOSITION

Ingredients	Concentration
Vancomycin	1.500mg
Colistin methane sulphonate	3.750mg
Trimethoprim	2.500mg
Nystatin	6250Units

INSTRUCTION FOR USE (per vial sufficient for 500ml medium)

Rehydrate the contents of 1 vial aseptically with 2 ml sterile distilled water. Mix well and aseptically add it to THAYER MARTIN MEDIUM BASE for 440 ml of medium aseptically add 50ml sterile lysed blood and one vial of V.C.N.T Supplement along with one vial of Vitamino Growth Supplement, 2% Haemoglobin Solution (250ml) can be used instead of sterile lysed blood in 250ml of medium.

In GC AGAR BASE / TMV 116 - GC AGAR BASE (VEG.) for 250 ml of medium aseptically add 250 ml of 2% Haemoglobin Solution and GC Supplement w/Antibiotics, one vial of Vitamino Growth Supplement w/ Antibiotics or Yeast

Autolysate Supplement. If desired V.C.N.T Supplement can be used along with GC Supplement w/ Antibiotics for additional selectivity.

In TRANSGROW MEDIUM BASE for 440 ml of medium aseptically add 50 ml of sterile 2% Haemoglobin Solution and one vial of V.C.N.T Supplement along with one vial of Vitamino Growth Supplement.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative MediStar GmbH Sankt Augustin 19 50153 Aachen, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

