

CM 23921 – XLT4 SUPPLEMENT

INTENDED USE

For isolation of non-typhi Salmonella.

COMPOSITION

Ingredients	Concentration
(40ml per vial contains Tergitol 4 - 27%)	-
Tergitol 4	10.800g
Distilled water	40ml

(per vial sufficient for 1000 ml medium)

INSTRUCTION FOR USE

Warm up XLT4 Supplement to room temperature and add 4.6ml to 1000 ml of XLT4 AGAR BASE. Heat to boiling to dissolve the medium completely. Avoid overheating. DO NOT AUTOCLAVE. Mix well and pour into sterile petri plates.

STORAGE

Vials should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative WuXi AppTec Suzhou IVD, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only