

CM 23923 – YERSINIA SELECTIVE SUPPLEMENT

INTENDED USE

For selective isolation of *Yersinia enterocolitica*.

COMPOSITION

Ingredients	Concentration
Cefsulodin	7.500mg
Triclosan(Irgasan)	2mg
Novobiocin	1.250mg

(per vial sufficient for 500ml medium)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 2 ml of sterile distilled water and 1 ml of ethanol. Mix gently to dissolve the contents completely and aseptically add to 500 ml of sterile, molten, cooled (45-50°C) YERSINIA SELECTIVE AGAR BASE (CIN AGAR) / YERSINIA SELECTIVE AGAR BASE, W/ 1.2% AGAR / YERSINIA SELECTIVE BROTH BASE. Mix well and pour into sterile petri plates.

STORAGE

Vial should be stored in sealed container at 2-8°C.

 GMP Good Manufacturing Practices Certified	 IVD For In Vitro Diagnostic Use	 QTY. Quantity	 LOT/ B. NO. Lot / Batch Number	 REF Catalogue Number	 Manufacturer
 Temperature Unit	 EC REP Authorized Representative Müller-Guttmann GmbH Buckenhof 10 41161 Mönchen-Gladbach, Germany	 European Conformity	 QR Code	 Consults Instructions for Use	 Best Before

NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only